

OPERATIONS MANUAL

MedRx Video Otoscope Software

Bringing Video Otoscopy To Your Computer



MedRx[®]

Let Our Image Enhance Your Image

Contents

Video Otoscope Software


Software Installation for the Stand Alone VO	4
USB Device & Driver Installation	6
Software Configuration	8
Adjusting Video Quality	10
Taking Pictures	11
The Capture Screen	12
The Main Screen	13
The View Screen	14
Saving Images.....	15



www.medrx-int.com

Manufacturer: MedRx, Inc.
1200 Starkey Rd., #105, Largo FL 33771, U.S.A.
Toll Free: (888) 392-1234 • (727) 584-9600
Fax: (727) 584-9602 • E-mail: medrx@medrx-usa.com
www.medrx-usa.com

MedRx's Authorized Representative in Europe: Emergo Europe
(Regulatory affairs only)
Molenstraat 15, 2513 BH The Hague, The Netherlands



#0086

Distributor: MAICO Diagnostics GmbH
MedRx International Sales
Sickingenstr. 70-71, 10553 Berlin, Germany
Tel.: +49 30/70 71 46-50, Fax: -99
E-mail: medrx-sales@maico.biz
Web: www.medrx-int.com

The MedRx Video Otoscope Software


Software Installation

DO NOT PLUG THE USB CABLE INTO YOUR COMPUTER UNTIL INSTRUCTED TO DO SO.


The MedRx Video Otoscope software may be incorporated into any AVANT™ software package. It is also available as a stand-alone application. When bundled with the software packages, the Video Otoscope software is automatically installed. In this case, go directly to the Driver Installation section on page 5.


Stand Alone Video Otoscope Software Installation

1. Insert the MedRx Video Otoscope CD-ROM into the CD drive. Wait until the **Setup program starts**.
 - If the Setup program does not automatically start, go to **my computer**, double click on **CD**.
2. On the Setup screen, choose **Install OtoScope**.

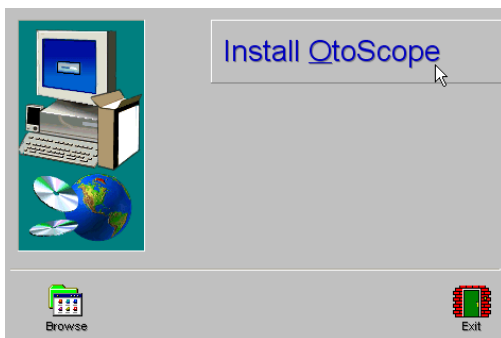


3. This is the Welcome screen.
 - To continue, Click **Next**.

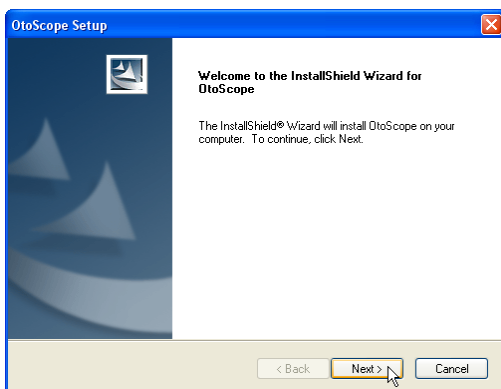

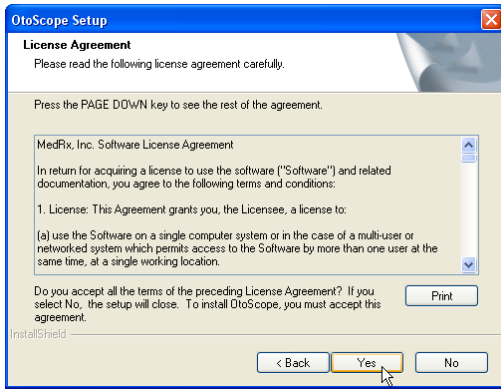


4. Read the Software License Agreement. This important document defines the acceptable usage of the Otoscope software.

After reading the Agreement, click **Yes**.

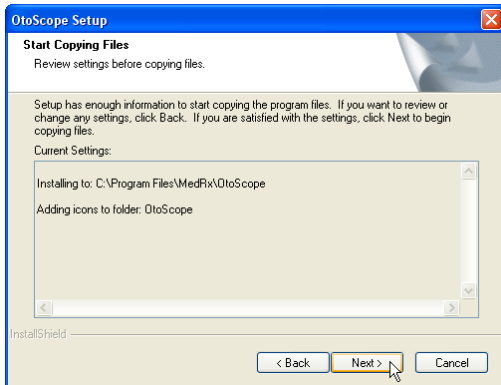


5. This screen summarizes the installation configuration.
 - To continue, click **Next**.
 - To make changes, click **Back**.

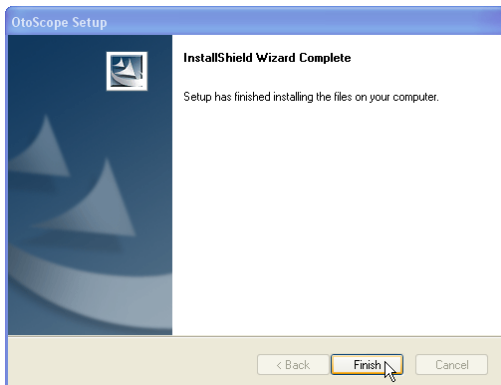

6. When the installation is complete,
 - Click **Finish**.


USB Device and Driver Installation

Connect the USB cable from the USB Video Capture Device to your computer as shown below.

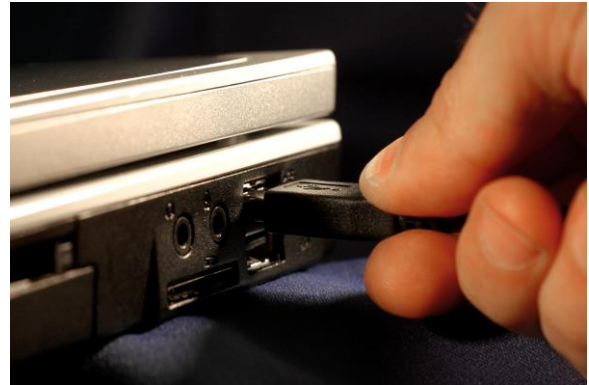


Shown Above - Typical USB Capture Cables


1. Insert the Video Driver CD-ROM into the CD drive.

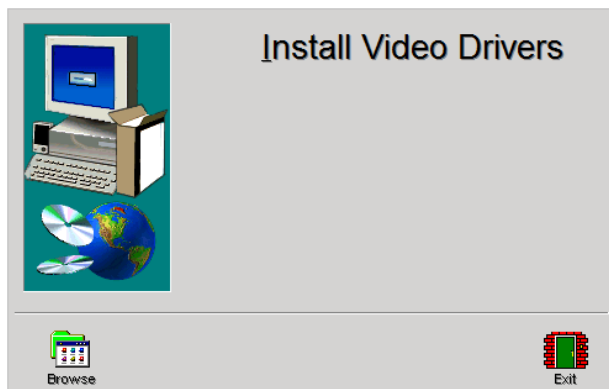


2. On the Video Driver Setup screen, click **Install Video Drivers**.

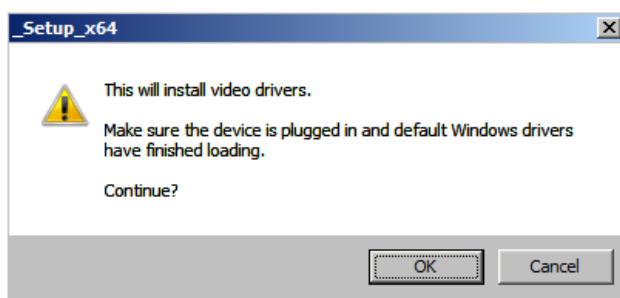


3. When this screen appears, click **Ok**.

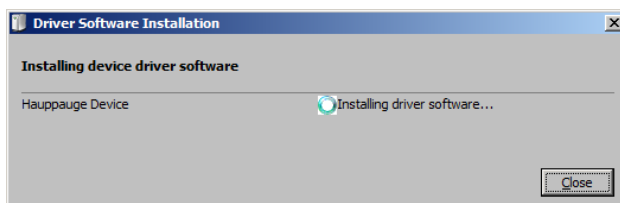

4. When this window appears and device is "Ready to use", click **Close**.

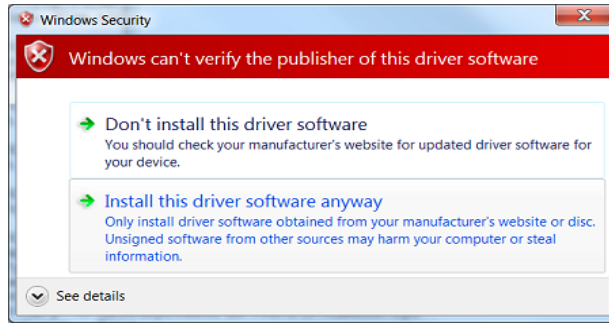

5. If this warning screen appears, click **Install this driver software anyway**.

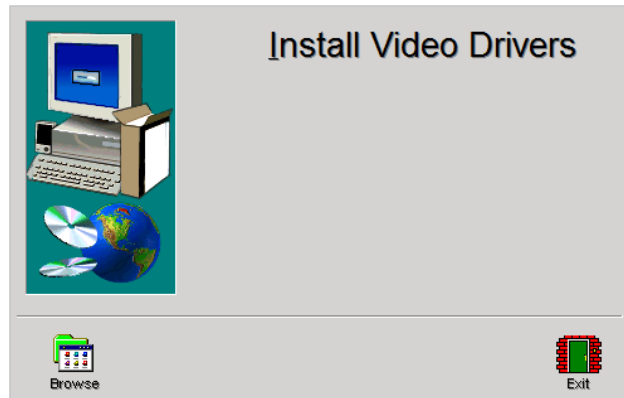

6. On Setup Window click **Exit**

Software Configuration

The MedRx Video Otoscope software can run stand-alone, through Avant Family Device software or through the NOAH™ System or the TIMS® Office System.

Launching the Software

Stand Alone or AVANT Family

1. Double click the MedRx OtoScope icon or select Otoscope from any **AVANT Family device** shortcut on your Windows Desktop.



NOAH and TIMS

1. Launch NOAH and select **AVANT Family device** icon.




Configuring the OtoScope Software

Once the drivers are installed, you must configure the software to recognize the video capture device as follows:

For Stand Alone software

From the main screen of the Otoscope software,


- Click **File**.
- Click **Video Setup**.

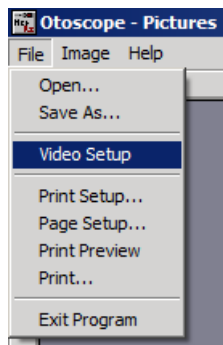


For Avant Family software

From the main screen of the Otoscope software,

- Click **File**.
- Click **Advanced Options**

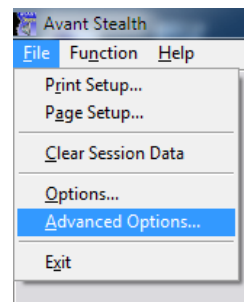


- Then Click **Video tab**

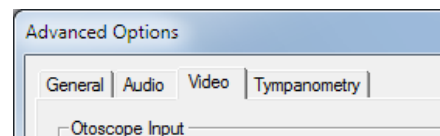


Both are continued on next page

For Stand Alone and Avant Family software

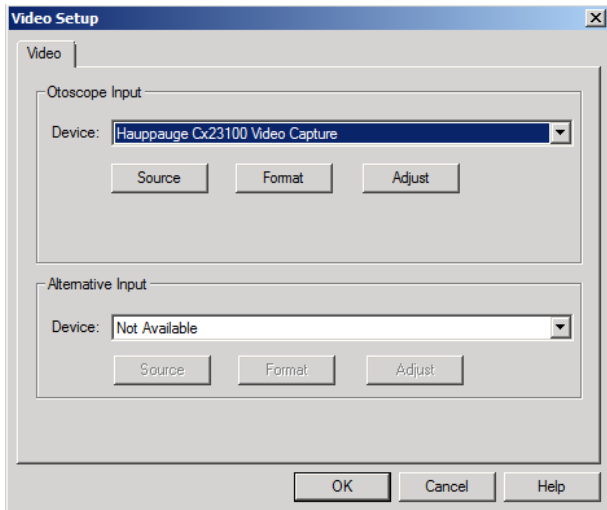


1. From the **Video Tab**:
 - Click the **Device** pull-down.
 - Select the name of your Video Capture Device
2. In the same screen, click the **Source** button.

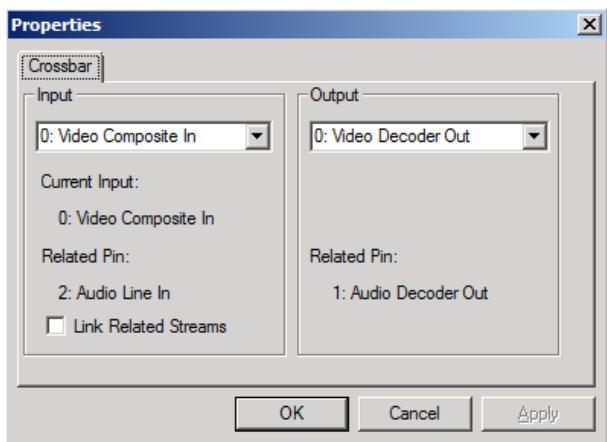


3. In the Properties window, use the pull-downs to verify the following settings:
 - **Input = Video Composite In.**
 - **Output = Video Decoder Out.**
4. Click **OK**.

Adjusting Video Quality

It is sometimes desirable to adjust the video quality of the OtoScope software. To do this, follow the steps below:

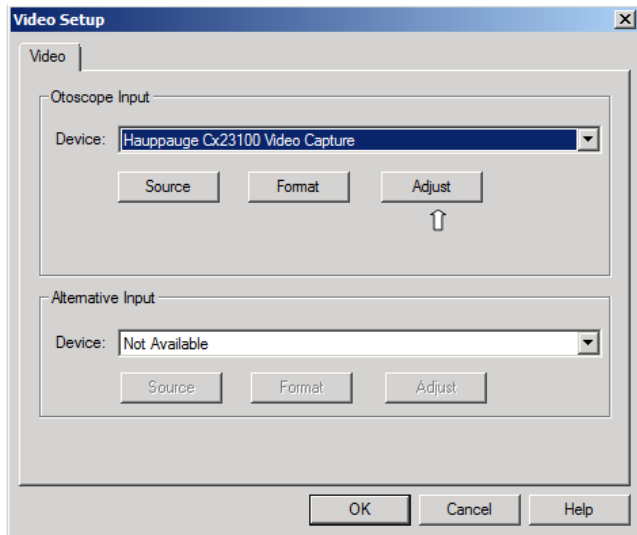

5. From the **Video Tab**:
 - Click **File**.
 - Click **Video Setup**.
 - Click the **Adjust** button.

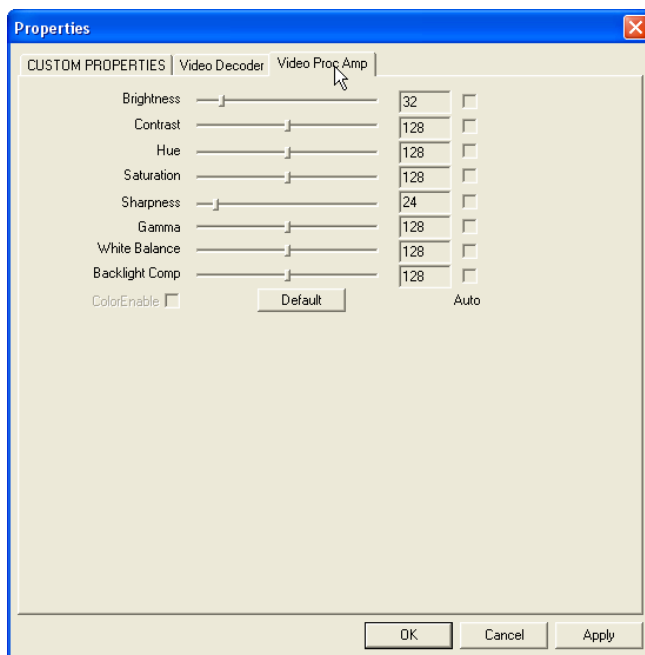


1. Click on the **Video Proc Amp** Tab.
2. Adjust the sliders as needed.
3. Click **OK** to complete.
4. Repeat as needed to optimize the image on your particular system.

Taking Pictures

Within NOAH™ or stand-alone

*Always Use a clean speculum tip for each new patient.
Refer to "Set-Up Manual" for additional cleaning information.*

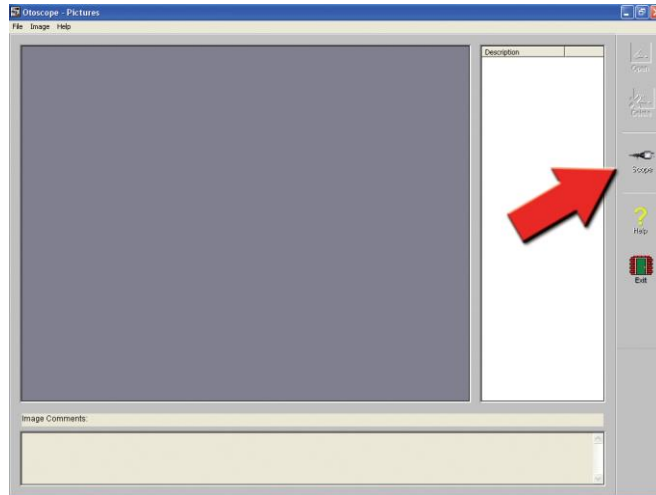




Click on the **Scope Icon** on the **Main Screen**.



The **Capture Screen** Appears.

The Capture Screen

Within NOAH™ or stand-alone

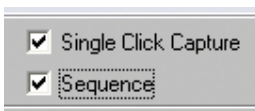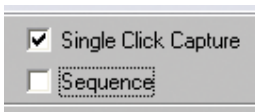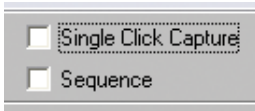

There are three ways to take pictures:

1. If you do not check either box:

When no items are selected, **click on the Capture Icon to take one picture, the Main Screen** appears.

2. If you check Single Click Capture.

When you click the **Capture Icon** instead of immediately taking a picture, the program switches to the **Single Click mode**.

All the icons will turn gray on the screen, indicating that they are inactive.

You can now **take a picture by pressing any key** on the keyboard or clicking the left mouse button, the main Screen appears.

3. Or if you check both boxes.

When you click the **Capture Icon** instead of immediately taking a picture all the icons will turn gray, indicating that they are inactive.

You can now **take as many pictures as you would like in a row without returning to the Main Screen by pressing any key** on the keyboard or the left mouse button.

To end the Single Click/Sequence mode, press the **Esc key** or the **right mouse button**.

Press the **Back** button to **return to the Main Screen** to work with your pictures.



Naming your pictures

Use the **pull-down menu at the top of your screen**. Select from the list provided or highlight box and type in any name.


The Main Screen

Within NOAH™ or stand-alone



The Main Screen shows a list and preview of the captured images.

An image in the list can be renamed by selecting it, clicking the right mouse button, and choosing the Rename option from the menu.

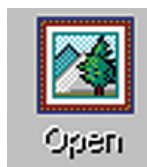

Open Icon:

Opens the selected picture in the **View Screen** where you can enhance it.



Scope Icon:

Click to take a new picture.

The View Screen

Within NOAH™ or stand-alone



This screen allows you to enhance your pictures.

Tool Bar



Use this button to enlarge the image.



Use this button to restore the image to its the original size.



Use this button to maximize the image so that it occupies all available screen space.



Use this button to undo the last operation performed on the image.

Enhancing



Sharpen

Use this function to emphasize the details on the image.



Smooth

Use this function to blur the details on the image.



Shadow


Use this function to create a shadow effect on the image. Best when used with monochrome pictures.




Deinterlace

Use this function to reduce motion artifacts.

Saving Images

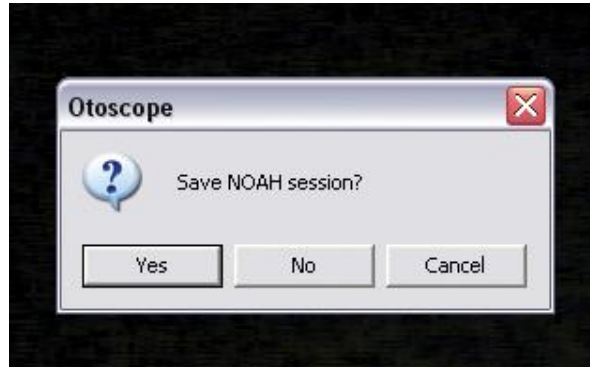

Using NOAH: When you exit the program you will get a box that asks "do you want to Save NOAH session?". Click **Yes**.

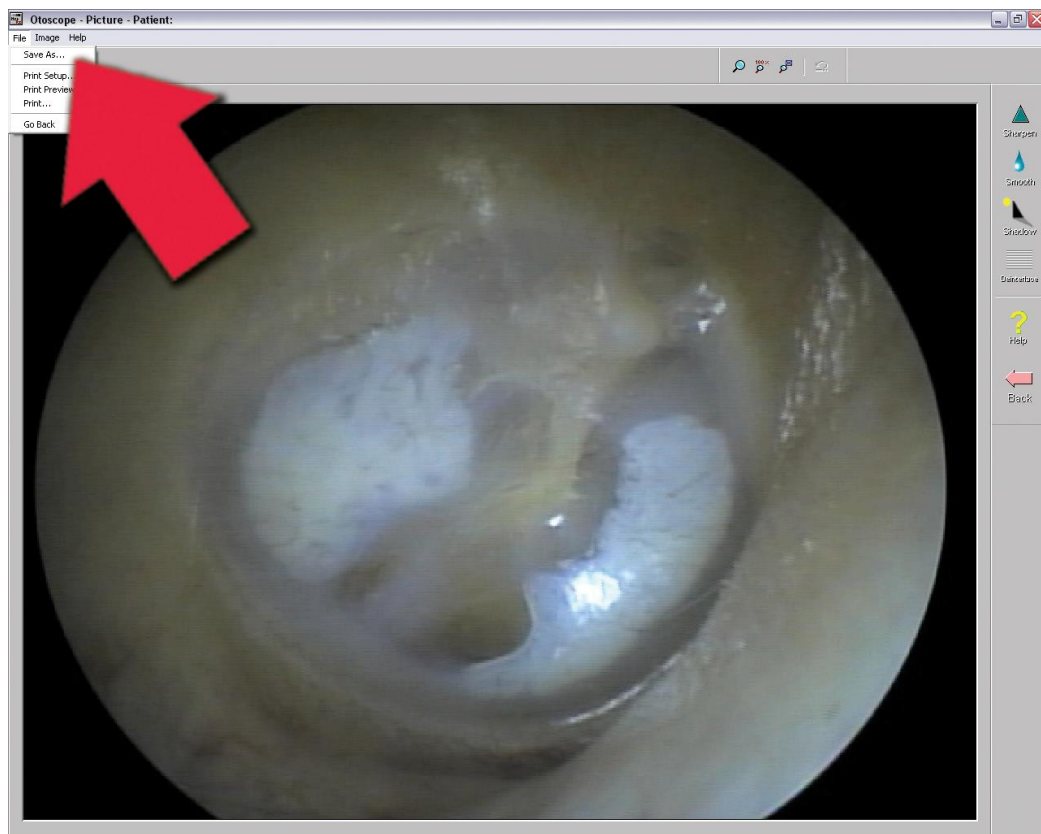

Not using NOAH: from the View Screen go up to the File Menu and choose Save As. Save your images to your desktop in jpg or bmp formats.



www.medrx-int.com

Manufacturer:

MedRx, Inc.
1200 Starkey Rd., #105, Largo FL 33771, U.S.A.
Toll Free: (888) 392-1234 • (727) 584-9600
Fax: (727) 584-9602 • E-mail: medrx@medrx-usa.com
www.medrx-usa.com

**MedRx's Authorized
Representative in Europe:**



Emergo Europe
(Regulatory affairs only)
Molenstraat 15, 2513 BH The Hague, The Netherlands

Distributor:

MAICO Diagnostics GmbH
MedRx International Sales
Sickingenstr. 70-71, 10553 Berlin, Germany
Tel.: +49 30/70 71 46-50, Fax: -99
E-mail: medrx-sales@maico.biz
Web: www.medrx-int.com